Würfel-Pessar

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CE 0483

Made in Germany

MEDESIGN

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	Deutsch	4
	English	11
	Français	19
	Italiano	27
	Español	35
	Türkçe	43
	Polski	51
	Русский	59
	中文	67
	يَبْرُع	82

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Cube Pessary

Instructions for use

When is therapy with a Cube Pessary useful?

Cube Pessaries are used for treating certain types of incontinence (inability to control urination or defecation) as well as problems caused by descensus of the vagina, the uterus, the bladder or the intestine. Descents like this can be caused by e.g. hereditary weakness of the connective tissue, postmenopausal hormone deficiency or pregnancy and giving birth. Treatment with pessaries - combined with pelvic floor exercise and, if necessary, oestrogen enriched creams - can permanently improve conditions and allow changing to a smaller sized pessary or even totally obviate the need for a pessary.

How does a Cube Pessary work?

The concave sides of the Cube Pessary adhere to the vaginal wall like suction cups and the cube's edges stabilise the tissue. The medesign pessary repositions the organs and brings them into the anatomically correct position – counteracting a prolapse or descensus of the vagina, the uterus, the bladder or the intestine. Therapy with pessaries is an alternative to surgical intervention for the treatment of descensus uteri and incontinence.

Which pessary is the right one for me?

medesign's Cube Pessaries are offered in various sizes and models. Selecting the correct size is a prerequisite for ideal treatment and freedom from symptoms. Your gynaecologist will determine the right size for you. Select a pessary, which securely stays in the vaginal vault even when coughing and squeezing. Your gynaecologist will assess the fit of your pessary after a few weeks or months to see whether you can switch to a different size.

Cube Pessaries RV are perforated in order for them not to adhere too strongly and hence allow easier removal. Furthermore they allow for better drainage of

vaginal secretion. medesign's pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. If you should, despite this, notice a reaction, stop the use of the pessary and discuss the situation with your physician. The pessaries are flexible and can thus be folded and squeezed, making insertion and removal especially easy. The pessary may be used by one patient only.

How do I insert the Cube Pessary?

You should insert the Cube Pessary into the vagina in the mornings and remove it in the evenings in order for the sensitive vaginal tissue to be able to recover overnight. In certain cases the pessary may remain inserted during the night or be replaced by a smaller cube at night. Instead of changing the cube themselves, many older patients leave this to their physician. Removal and reinsertion is necessary every 4 to 8 weeks. Please follow your doctor's instructions for this.

Wash your hands prior to inserting the pessary. To ease insertion it is recommended that a lubricant, such as Ladysoft, be used. If needed, the edges of the cube may be coated with a cream or gel containing oestrogen.

Now insert the pessary into the vagina like a tampon with the string hanging down. It may be helpful to place one leg e.g. on the edge of your bed. Some women find it easier to insert the cube in a crouching or lying position. You could also lean against a wall with your legs slightly spread and then insert the pessary. Try out which method is best for you.

Push the cube as far into the vagina as possible or into the position shown to you by your physician. The cube is placed correctly if you cannot feel any pressure.

How do I remove the Cube Pessary?

You should also wash your hands prior to removal. Thoroughly lubricate your vaginal area with Ladysoft Lubricant. Use the attached string to pull the cube downwards until you feel a slight resistance from the pelvic floor muscles. It is best to crouch down and pull the string into different directions. Now use your index or middle finger (do not use any other assistive equipment) to detach the cube's edges from the vaginal wall and remove the Cube Pessary from your vagina by pulling the string. Do not use any other equipment for removing the Cube Pessary.

What do I do if the string tears?

Check the strength of the withdrawal string before each use of the pessary.

You can use your fingers to remove the pessary, in the exceptional case of a torn string. If you are not able to remove the pessary, let your physician remove it within the next few days.

Removing the cube:

Always use your fingers to detach the cube's edges from the vaginal wall, allowing for air to enter. Otherwise you risk tearing of the retrieval string.

Pessary Care

Clean the Cube Pessary with lukewarm water and soap or a mild washing lotion such as Femsoft after removal. Subsequently rinse and leave to dry. Disinfection is usually not necessary. Wash your hands again after cleaning the pessary.

For added support for the cleaning and care of your intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion with whey and stinging nettle as a basis and a low pH-value (3.8–4). Subsequently wash your hands again.

**Do not use chemicals for cleaning.
Do not use household or dish detergents.
It is not necessary to boil the pessary.
Do not put it in the microwave.**

Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

**Do not use if you are oversensitive to silicone.
So far no allergic reactions known.**

Infections of the vagina and the cervix are contraindications for pessary treatment.

Please note

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Uninterrupted period of application without medical supervision: 29 days maximum.
- Maximum period of application with medical supervision: 2 - 3 cycles of 29 days maximum.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and/or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The pessary has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the pessary with clear water.
- The periods of application have to be documented by a physician.
- Do not use during menstruation and never use the pessary together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

Additional products

Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

Usable until

LOT

Date of construction

Producer

CE 0483 CE-mark with identification number of the notified body

Item number (MED...)

Attention

Read and follow the instructions for use

Single patient – multiple use

Cube Pessary**Size****Item No.**

Size 0	25 mm Feed size	MED1000078
Size 1	29 mm Feed size	MED1000079
Size 2	32 mm Feed size	MED1000080
Size 3	37 mm Feed size	MED1000081
Size 4	41 mm Feed size	MED1000082
Size 5	45 mm Feed size	MED1000083

Cube Pessary RV**Size****Item No.**

Size 0	25 mm Feed size	MED1000645
Size 1	29 mm Feed size	MED1000644
Size 2	32 mm Feed size	MED1000643
Size 3	37 mm Feed size	MED1000642
Size 4	41 mm Feed size	MED1000641
Size 5	45 mm Feed size	MED1000640
Size 6	55 mm Feed size	MED1000949
Size 7	65 mm Feed size	MED1000950
Size 8	70 mm Feed size	MED1000951
Size 9	75 mm Feed size	MED1001177

Content:

1 pessary with instruction manual

1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

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medesign I.C. GmbH
medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell