# Siebschalen-Pessar

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CE 0483

Made in Germany

**M E D E S T G N**  
Dietramszell · Dubai · München

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	Deutsch .....	4
	English .....	11
	Français .....	19
	Italiano .....	27
	Español .....	35
	Türkçe .....	43
	Polski .....	51
	Русский .....	59
	中文 .....	67
	يَبْرَع .....	82

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# Sieve Bowl Pessary

Instructions for use

## When is therapy with a Sieve Bowl Pessary useful?

Sieve Bowl Pessaries are used for treating certain types of incontinence (inability to control urination or defecation) as well as problems caused by descensus of the vagina, the uterus, the bladder or the intestine. Descents like this can be caused by e.g. hereditary weakness of the connective tissue, postmenopausal hormone deficiency or pregnancy and giving birth. Treatment with pessaries – combined with pelvic floor exercise and, if necessary, oestrogen enriched creams – can permanently improve conditions and obviate the need for the pessary.

## How does a pessary work?

The medesign pessary repositions the organs and brings them into the anatomically correct position – counteracting a prolapse or descensus of the vagina, the uterus, the bladder or the intestine. Therapy with pessaries is an alternative to surgical intervention for the treatment of descensus uteri and incontinence.

## Which pessary is the right one for me?

Selecting the correct size is a prerequisite for ideal treatment and freedom from symptoms. medesign's Sieve Bowl Pessaries are offered in various sizes (50 - 100 mm) and models. The pessary, which is smallest in size, yet resting securely should be chosen.

Your gynaecologist will determine the right size for you. The pessary is inserted and you can check its correct position by moving, coughing and squeezing, whereupon the gynaecologist will check the fit of the pessary once again. The pessary, which is smallest in size, yet resting securely should be chosen. You should also make sure that you are able to urinate without any problems.

Your gynaecologist will assess the fit of your pessary after a few weeks or months to see whether you can switch to a different size. The various sizes allow for

individual fitting. Treatment is often carried out with increasing pessary sizes in order to achieve better results.

medesign pessaries are made from flexible, tissue tolerant silicone, which is especially developed for medical use. If you should, despite this, notice a reaction, stop the use of the pessary and discuss the situation with your physician. The pessaries are flexible and can thus be folded and squeezed, making insertion and removal especially easy. The holes in the Sieve Bowl Pessary allow for better drainage of vaginal secretion. The pessary may be used by one patient only.

### How do I insert the Sieve Bowl Pessary?

Pessaries can easily be inserted on your own. Check the Sieve Bowl Pessary for intactness, cleanliness, discolouration, damage and wear marks. The pessary has to be replaced by a new one should damage or changes on the surface be visible.

You should insert the Sieve Bowl Pessary into the vagina in the mornings and remove it in the evenings in order for the sensitive vaginal tissue to be able to recover overnight. In certain cases the pessary may remain inserted during the night or be replaced by a smaller Sieve Bowl Pessary at night. Instead of changing the pessary themselves, many older patients leave this to their physician. Removal and reinsertion is necessary every 4 to 8 weeks. Please follow your doctor's instructions for this.

Wash your hands prior to inserting the pessary. To ease insertion it is recommended that a lubricant, such as Ladysoft, be used. If needed, the Sieve Bowl Pessary may be coated with a cream or gel containing oestrogen.

Now insert the folded/squeezed Sieve Bowl Pessary into the vagina. It may be helpful to place one leg e.g. on the edge of your bed. Some women find it easier to insert the ring in a crouching or lying position. You could also lean against a wall with your legs slightly spread and then insert the pessary. Try out which method is best for you.

Push the pessary into the position shown to you by your physician. Make sure that you place the folded Sieve Bowl Pessary in the back vaginal vault and subsequently push the bowl up towards the front. The pessary is placed correctly if you cannot feel any pressure. If the Sieve Bowl Pessary is not placed correctly it cannot fulfil its supporting function and will slide out under strain.

## How do I remove the Sieve Bowl Pessary?

You should also wash your hands prior to removal. Thoroughly lubricate your vaginal area with Ladysoft Lubricant. Grab the bowl with your index finger and pull the pessary out of your vagina. Do not use any other equipment for removing the Sieve Bowl Pessary. Subsequently wash your hands again.

## What do I do if i cannot remove the pessary?

Do not worry if you have difficulties removing the pessary – these soft pessaries can stay in place for a few weeks without any negative effects. Your gynaecologist will remove the pessary at your next visit (within 4 weeks).

## Pessary Care

Clean the Sieve Bowl Pessary with lukewarm water and soap or a mild washing lotion such as Femsoft after removal. Subsequently rinse and leave to dry. Disinfection is usually not necessary. Wash your hands again after cleaning the pessary.

For added support for the cleaning and care of you intimate area, use e.g. Femsoft Intimate Wash – a specially developed washing lotion with whey and stinging nettle as a basis and a low pH-value (3.8 - 4). Subsequently wash your hands again.

- Do not use chemicals for cleaning.**
- Do not use household or dish detergents.**
- It is not necessary to boil the pessary.**
- Do not put it in the microwave.**

## Disposal

medesign pessaries are made from certified medical silicone and can be disposed of in household waste. The silicone is not biodegradable and should therefore not be disposed of in the toilet.

**Infections of the vagina and the cervix are contraindications for pessary treatment.**

**Please note**

- Read and follow instructions for use.
- Use according to instructions from healthcare professionals.
- Uninterrupted period of application without medical supervision: 29 days maximum.
- Maximum period of application with medical supervision: 2 - 3 cycles of 29 days maximum.
- Repeated use with the same patient possible.
- Tolerance and correct placement of the pessary have to be secured through ultrasound and/or consultation.
- The product has to be checked for intactness and cleanliness of the surface prior to every application. The pessary has to be replaced by a new one should changes on the surface appear.
- The pessary has to be cleaned after every application – using lukewarm water and soap and subsequently rinsing the pessary with clear water.
- The periods of application have to be documented by a physician.
- Do not use during menstruation and never use the pessary together with a tampon or without medical consultation.
- Do not use during pregnancy or if you think you might be pregnant.
- Do not use if you have a bladder or urinary tract infection or inflammation of the vagina.
- medesign pessaries are latex-free.

## Additional products

Ladysoft Lubricant: eases insertion of the pessary

Femsoft Intimate Wash: gentle cleansing of the intimate area

Femsoft Intimate Cream: nourishes and protects the intimate area

Usable until

LOT

Date of construction

Producer

Item number (MED...)

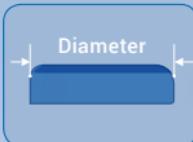
Attention

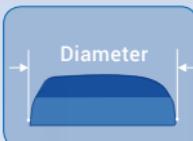
Read and follow the instructions for use

CE-mark with identification number  
of the notified body

**Sieve Bowl Pessary****Size**

$\varnothing$ 50 mm	MED1001533
$\varnothing$ 55 mm	MED1000110
$\varnothing$ 60 mm	MED1000111
$\varnothing$ 65 mm	MED1000112
$\varnothing$ 70 mm	MED1000113
$\varnothing$ 75 mm	MED1000114
$\varnothing$ 80 mm	MED1000115
$\varnothing$ 85 mm	MED1000116
$\varnothing$ 90 mm	MED1000117
$\varnothing$ 95 mm	MED1001534
$\varnothing$ 100 mm	MED1001535

**Sieve Bowl Pessary EF****Size**

$\varnothing$ 50 mm	MED1001536
$\varnothing$ 55 mm	MED1001537
$\varnothing$ 60 mm	MED1001538
$\varnothing$ 65 mm	MED1001539
$\varnothing$ 70 mm	MED1001540
$\varnothing$ 75 mm	MED1001541
$\varnothing$ 80 mm	MED1001542
$\varnothing$ 85 mm	MED1001543
$\varnothing$ 90 mm	MED1001544
$\varnothing$ 95 mm	MED1001545
$\varnothing$ 100 mm	MED1001546

**Content:**

1 pessary with instruction manual

1 Ladysoft Lubricant with instruction manual

If you have any questions regarding application or would like to share your experiences, please contact us.

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**medesign I.C. GmbH**
**medesign I.C. GmbH · Dietramszeller Str. 6 · D-83623 Dietramszell**