

Edition 2015

Imprint

Edition 2015

This documentation is a free information service of the medesign I.C. GmbH, Dietramszell – Germany

CEO Register court VAT number Copyright Lothar Engel District court munich, HRB 91285 DE 129411976 medesign I.C. GmbH

Reproduction by written permission only. All information subject to change. Originals can production conditions differ from illustration. Features and specifications are subject to change.

Contents

5

Ring Pessary (thin) 4 Widely used in the treatment of uterine descent and prolapse vaginae and the associated urinary incontinence

- Ring Pessary (thick) Widely used in the treatment of uterine descent and prolapse vaginae and the associated urinary incontinence
 - Supported at a slight vaginal and uterine prolapse 6
 - Sieve Bowl Pessary EF 7 Supported at a slight vaginal and uterine prolapse
 - Urethra Pessary 8 Supports the urethra and improves the sealing function.
 - Urethra Sieve Pessary 9 Provides greater descent at a better maintenance
 - Cerclage Pessary 10 Enables the prevention of cervical insufficiency
 - Cerclage Pessary Hamann & Jorde 11 For prophylaxis and treatment of an emerging premature birth
- For the treatment of stress incontinence and complaints caused by a reduction of the vagina, uterus and bladder
 - Club Pessary 13 For treating large prolapse. Prevented by its stem jamming and slipping
 - Cube Pessary 14 For most forms of genital cutting
- Cube Pessary RV 15 Good adhesion and is easy to use. Can be used with most types of genital cutting. Through the holes, the vacuum is reduced
 - Tandem Cube Pessary
 16

 Suitable for treatment of extreme vaginal and uterine prolapse
 16
 - T Pessary Type B 19 For special treatment of cystocele and rectocele
 - T Pessary Type R 20 For special treatment of cystocele
 - Incontinence Tampon Pessary 22 For special treatment of cystocele and rectocele
 - Pessary Fitting Sets 24

Ring Pessary (thin)

Description

Slight prolapse (descensus) and urinary incontinence (aconuresis) can be successfully treated without surgery. The Ring Pessary has been successfully tried and tested. It is used for treating uterine prolapse (descensus uteri) and vaginal prolapse as well as urinary incontinence (aconurensis). Ring Pessaries are made from medical silicone and are available in various sizes in order to offer each individual patient the best possible comfort.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Ring Pessary thin	Ø 45 mm	1 pc	MED1001442	4250201201942
Ring Pessary thin	Ø 50 mm	1 pc	MED1000100	4250201201959
Ring Pessary thin	Ø 55 mm	1 pc	MED1000101	4250201201966
Ring Pessary thin	Ø 60 mm	1 pc	MED1000102	4250201201973
Ring Pessary thin	Ø 65 mm	1 pc	MED1000103	4250201201980
Ring Pessary thin	Ø 70 mm	1 pc	MED1000104	4250201201997
Ring Pessary thin	Ø 75 mm	1 рс	MED1000105	4250201202000
Ring Pessary thin	Ø 80 mm	1 pc	MED1000106	4250201202017
Ring Pessary thin	Ø 85 mm	1 pc	MED1000107	4250201202024
Ring Pessary thin	Ø 90 mm	1 pc	MED1000108	4250201202031
Ring Pessary thin	Ø 95 mm	1 рс	MED1000109	4250201202048
Ring Pessary thin	Ø 100 mm	1 pc	MED1000434	4250201202055

Dimensions

Top View







Ring Pessary (thick)

Description

Slight prolapse (descensus) and urinary incontinence (aconuresis) can be successfully treated without surgery. The Ring Pessary has been successfully tried and tested. It is used for treating uterine prolapse (descensus uteri) and vaginal prolapse as well as urinary incontinence (aconurensis). The larger circumference of the ring increases the contact area and therefore minimizes the risk of pressure sores (pressure necrosis). Ring Pessaries are made from medical silicone and are available in various sizes in order to offer each individual patient the best possible comfort.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Ring Pessary thick	Ø 50 mm	1 pc	MED1000817	4250201201300
Ring Pessary thick	Ø 55 mm	1 pc	MED1000818	4250201201317
Ring Pessary thick	Ø 60 mm	1 pc	MED1000819	4250201201324
Ring Pessary thick	Ø 65 mm	1 pc	MED1000820	4250201201331
Ring Pessary thick	Ø 70 mm	1 pc	MED1000821	4250201201348
Ring Pessary thick	Ø 75 mm	1 pc	MED1000822	4250201201355
Ring Pessary thick	Ø 80 mm	1 pc	MED1000823	4250201201362
Ring Pessary thick	Ø 85 mm	1 pc	MED1000824	4250201201379
Ring Pessary thick	Ø 90 mm	1 pc	MED1000825	4250201201386
Ring Pessary thick	Ø 95 mm	1 pc	MED1000826	4250201201393
Ring Pessary thick	Ø 100 mm	1 рс	MED1000827	4250201201409

Dimensions

Top View



Side View



Sieve Bowl Pessary

Description

medesign's Sieve Bowl Pessary helps with slight vaginal and/or uterine prolapse. It is easy to use. The soft medical silicone makes it easily foldable and therefore simplifies insertion and removal. Just like the Ring Pessary, it can be easily placed in the right position. It can also be used reversed. medesign's Sieve Bowl Fitting Set helps to determine the correct size of the pessary.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Sieve Bowl Pessary	Ø 50 mm	1 pc	MED1001533	4250201202864
Sieve Bowl Pessary	Ø 55 mm	1 pc	MED1000110	4250201201416
Sieve Bowl Pessary	Ø 60 mm	1 pc	MED 1000111	4250201201423
Sieve Bowl Pessary	Ø 65 mm	1 pc	MED1000112	4250201201430
Sieve Bowl Pessary	Ø 70 mm	1 pc	MED1000113	4250201201447
Sieve Bowl Pessary	Ø 75 mm	1 pc	MED1000114	4250201201454
Sieve Bowl Pessary	Ø 80 mm	1 pc	MED1000115	4250201201461
Sieve Bowl Pessary	Ø 85 mm	1 pc	MED1000116	4250201201478
Sieve Bowl Pessary	Ø 90 mm	1 pc	MED1000117	4250201201485
Sieve Bowl Pessary	Ø 95 mm	1 pc	MED1001534	4250201202871
Sieve Bowl Pessary	Ø 100 mm	1 pc	MED1001535	4250201202888

Dimensions

Top View







Sieve Bowl Pessary EF

Description

medesign's Sieve Bowl Pessary helps with slight vaginal and/or uterine prolapse. It is easy to use. The soft medical silicone makes it easily foldable and therefore simplifies insertion and removal. Just like the Ring Pessary, it can be easily placed in the right position. It can also be used reversed. It is more curved and therefore stiffer than the normal Sieve Bowl Pessary.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Sieve Bowl Pessary EF	Ø 50 mm	1 pc	MED1001536	4250201202895
Sieve Bowl Pessary EF	Ø 55 mm	1 pc	MED1001537	4250201202901
Sieve Bowl Pessary EF	Ø 60 mm	1 pc	MED1001538	4250201202918
Sieve Bowl Pessary EF	Ø 65 mm	1 pc	MED1001539	4250201202925
Sieve Bowl Pessary EF	Ø 70 mm	1 pc	MED1001540	4250201202932
Sieve Bowl Pessary EF	Ø 75 mm	1 pc	MED1001541	4250201202949
Sieve Bowl Pessary EF	Ø 80 mm	1 рс	MED1001542	4250201202956
Sieve Bowl Pessary EF	Ø 85 mm	1 pc	MED1001543	4250201202963
Sieve Bowl Pessary EF	Ø 90 mm	1 pc	MED1001544	4250201202970
Sieve Bowl Pessary EF	Ø 95 mm	1 pc	MED1001545	4250201202987
Sieve Bowl Pessary EF	Ø 100 mm	1 pc	MED1001546	4250201202994

Dimensions

Top View



Side View



Urethra Pessary

Description

medesign's Urethra Pessary is used successfully in the conventional treatment for urinary incontinence (aconuresis). The Urethra Pessary, a flexible silicone ring with a thickening bulge (calvaria), has been successfully tried and tested. The thickening of the upper edge supports the lowered urethra during the increased pressure from the bladder e.g. when coughing, it prevents the opening of the upper urethra.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Urethra Pessary	Ø 45 mm	1 pc	MED1000060	4250201201508
Urethra Pessary	Ø 50 mm	1 pc	MED1000437	4250201201515
Urethra Pessary	Ø 55 mm	1 pc	MED1000061	4250201201522
Urethra Pessary	Ø 60 mm	1 pc	MED1000062	4250201201539
Urethra Pessary	Ø 65 mm	1 pc	MED1000063	4250201201546
Urethra Pessary	Ø 70 mm	1 pc	MED1000064	4250201201553
Urethra Pessary	Ø 75 mm	1 pc	MED1000065	4250201201560
Urethra Pessary	Ø 80 mm	1 pc	MED1000066	4250201201577
Urethra Pessary	Ø 85 mm	1 pc	MED1000067	4250201201584
Urethra Pessary	Ø 90 mm	1 pc	MED1000068	4250201201591
Urethra Pessary	Ø 95 mm	1 pc	MED 1000070	4250201201607
Urethra Pessary	Ø 100 mm	1 pc	MED1000069	4250201201614

Dimensions

Top View







Urethra Bowl Pessary

Description

medesign's Urethra Bowl Pessary is a combination of the Cerclage Pessary with a thickening bulge (calvaria), that is placed directly under the urethra. The thickening of the upper edge supports the lowered urethra during increased pressure from the bladder e.g. when coughing, it prevents the opening of the upper urethra. The robust body provides a stronger support than the normal Urethra Pessary at severe descensus. The larger contact area significantly minimizes the pressure on the tissue. It is recommended to remove the Urethra Bowl Pessary at night.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Urethra Bowl Pessary	Ø 55 mm	1 pc	MED1000071	4250201201638
Urethra Bowl Pessary	Ø 60 mm	1 pc	MED1000072	4250201201645
Urethra Bowl Pessary	Ø 65 mm	1 pc	MED1000073	4250201201652
Urethra Bowl Pessary	Ø 70 mm	1 pc	MED1000074	4250201201669
Urethra Bowl Pessary	Ø 75 mm	1 pc	MED 1000075	4250201201676
Urethra Bowl Pessary	Ø 80 mm	1 pc	MED 1000076	4250201201683
Urethra Bowl Pessary	Ø 85 mm	1 pc	MED 1000077	4250201201690

Dimensions

Top View



Side View



Cerclage Pessary

Description

Cerclage Pessaries are used to treat pregnant women with cervical insufficiency and therefore preventing late abortion or premature delivery. This riskless therapy is an alternative to surgical intervention. Both methods show a success rate of 80–90 %. Cerclage pessaries can be securely placed thanks to their bowl shape and curvature heights of 17, 21, 25 and 30 mm. Cerclage Pessaries are inserted by an gynecologist depending on the diagnostic findings after approximately week 16.



Material

Medical silicone, Silcopure soft Blue

Product name	Size	Contents	Article number	EAN-Code
Cerclage Pessary	Ø 32 / 65 / H 17 mm	1 pc	MED1000084	4250201201058
Cerclage Pessary	Ø 32 / 65 / H 21 mm	1 pc	MED1000050	4250201201065
Cerclage Pessary	Ø 32 / 65 / H 25 mm	1 рс	MED1000086	4250201201072
Cerclage Pessary	Ø 32 / 70 / H 17 mm	1 pc	MED1000087	4250201201089
Cerclage Pessary	Ø 32 / 70 / H 21 mm	1 pc	MED1000088	4250201201096
Cerclage Pessary	Ø 32 / 70 / H 25 mm	1 рс	MED1000089	4250201201102
Cerclage Pessary	Ø 35 / 65 / H 17 mm	1 рс	MED1000090	4250201201119
Cerclage Pessary	Ø 35 / 65 / H 21 mm	1 pc	MED1000091	4250201201126
Cerclage Pessary	Ø 35 / 65 / H 25 mm	1 pc	MED1000092	4250201201133
Cerclage Pessary	Ø 35 / 65 / H 30 mm	1 pc	MED1000093	4250201201140
Cerclage Pessary	Ø 35 / 70 / H 17 mm	1 рс	MED1000094	4250201201157
Cerclage Pessary	Ø 35 / 70 / H 21 mm	1 pc	MED1000095	4250201201164
Cerclage Pessary	Ø 35 / 70 / H 25 mm	1 pc	MED1000096	4250201201171

Dimensions

Top View







Cerclage Pessary Hamann & Jorde

Description

Cerclage Pessaries are used to treat pregnant women with cervical insufficiency and therefore prevent late abortion or premature delivery. This riskless therapy is an alternative to surgical intervention. The Cerclage & Support Pessary, patented by Dr Hamann and Dr Jorde, is used for prophylaxis and therapy of possible premature delivery. In the case of cervical insufficiency the Cerclage & Support Pessary takes the pressure off the cervix by supporting it, avoiding the risk of a cervical cerclage procedure.



Material

Medical silicone, Silcopure soft Blue

Product name	Size	Contents	Article number	EAN-Code
Cerclage Pessary H&J	Size I Ø 30 L 57, W 59, H 14	1 pc	MED1000097	4250201201188
Cerclage Pessary H&J	Size II Ø 35 L 70, W 76, H 14	1 pc	MED1000098	4250201201195
Cerclage Pessary H&J	Size III Ø 35 L 80, W 83, H 14	1 pc	MED 1000099	4250201201201

Dimensions

Top View



Side View



Cramer Pessary

Description

Cramer pessaries are designed for treating stress incontinence (involuntary leakage of urine) as well as conditions caused by descent of the vagina, uterus or bladder. The two thickenings on the upper edge of the pessary elevate the bladder neck, which prevent involuntary leakage of urine when the abdominal pressure is increased during movements such as e.g. coughing. A precondition to being allowed to use a Cramer Pessary is that no vaginal operation, that includes a vaginoplasty, has been performed, because the pelvic floor has a limited sustainability.



Material

Medical silicone, Silcopure soft Blue

Product name	Diameter	Contents	Article number	EAN-Code
Cramer Pessary	Ø 50 mm	1 pc	MED1000896	4250201202222
Cramer Pessary	Ø 55 mm	1 pc	MED1000897	4250201202239
Cramer Pessary	Ø 60 mm	1 pc	MED1000898	4250201202246
Cramer Pessary	Ø 65 mm	1 pc	MED1000899	4250201202253
Cramer Pessary	Ø 70 mm	1 pc	MED1000900	4250201202260
Cramer Pessary	Ø 75 mm	1 pc	MED1000901	4250201202277
Cramer Pessary	Ø 80 mm	1 pc	MED1000902	4250201202284
Cramer Pessary	Ø 85 mm	1 pc	MED1000903	4250201202291
Cramer Pessary	Ø 90 mm	1 pc	MED1000904	4250201202307

Dimensions

Top View







Club Pessary

Description

The Club Pessary is made from special medical silicone and is successfully used in treating patients with severe prolapses. The shaft prevents the ring from tilting and slipping. After about three weeks it is recommended that a smaller Club Pessary is inserted as the lower vaginal tissue shrinks relatively fast. Often the pessary can be replaced by a Cube Pessary after a couple of weeks. Insertion and removal of the Club Pessary is more difficult than with other pessaries.



Material

Medical silicone, Silcopure soft Transparent

Product name	Diameter	Contents	Article number	EAN-Code
Club Pessary	Ø 50 mm	1 pc	MED1000118	4250201201218
Club Pessary	Ø 55 mm	1 pc	MED 1000119	4250201201225
Club Pessary	Ø 60 mm	1 pc	MED1000120	4250201201232
Club Pessary	Ø 65 mm	1 pc	MED1000121	4250201201249
Club Pessary	Ø 70 mm	1 pc	MED1000122	4250201201256
Club Pessary	Ø 75 mm	1 pc	MED1000123	4250201201263
Club Pessary	Ø 80 mm	1 pc	MED1000124	4250201201270
Club Pessary	Ø 85 mm	1 pc	MED1000125	4250201201287
Club Pessary	Ø 90 mm	1 pc	MED1000433	4250201201294



Cube Pessary

Description

The Cube Pessary is indicated in patients with more severe complaints of genital decent where too weak to support other pessaries. medesign's Cube Pessaries are made from medical silicone and are known for their excellent vacuum effect and their extremely simple application method. The Cube Pessaries are also suitable for expanding the vagina before vaginal operations to stimulate local blood supply if coated with oestrogen cream.



Material

Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
Cube Pessary	Size 0 / 25 mm	1 pc	MED1000078	4250201201713
Cube Pessary	Size 1 / 29 mm	1 pc	MED1000079	4250201201720
Cube Pessary	Size 2 / 32 mm	1 pc	MED1000080	4250201201737
Cube Pessary	Size 3 / 37 mm	1 pc	MED1000081	4250201201744
Cube Pessary	Size 4 / 41 mm	1 pc	MED1000082	4250201201751
Cube Pessary	Size 5 / 45 mm	1 pc	MED1000083	4250201201768



Cube Pessary RV

Description

The Cube Pessary is indicated in patients with more severe complaints of genital decent where too weak to support other pessaries. medesign's cube pessaries are made from medical silicone and are known for their excellent vacuum effect and their extremely simple application method. The holes in the silicone provide a reduced vacuum, allowing an easier removal of the medesign Cube Pessary RV.



Material

Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
Cube Pessary RV	Size 0 / 25 mm	1 pc	MED1000645	4250201201775
Cube Pessary RV	Size 1 / 29 mm	1 pc	MED1000644	4250201201782
Cube Pessary RV	Size 2 / 32 mm	1 pc	MED1000643	4250201201799
Cube Pessary RV	Size 3 / 37 mm	1 pc	MED1000642	4250201201805
Cube Pessary RV	Size 4 / 41 mm	1 pc	MED1000641	4250201201812
Cube Pessary RV	Size 5 / 45 mm	1 pc	MED1000640	4250201201829
Cube Pessary RV	Size 6 / 55 mm	1 pc	MED1000949	4250201201836
Cube Pessary RV	Size 7 / 65 mm	1 pc	MED1000950	4250201201843
Cube Pessary RV	Size 8 / 70 mm	1 pc	MED1000951	4250201201850
Cube Pessary RV	Size 9 / 75 mm	1 pc	MED 1001177	4250201201867



Tandem Cube Pessary

Description

medesign's Tandem Cube Pessary consists of two connected cube pessaries and is used for severe vaginal and uterine prolapse. The concave surfaces of the Cube Pessary create an excellent vacuum. medesign's Tandem Cube Pessary can also help with stress incontinence. In this case, the bottom cube supports the urethra while the second cube ensures good retention.



Material

Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
Tandem Cube Pessary	1-0 / 29-25 mm	1 pc	MED1001071	4250201201881
Tandem Cube Pessary	2-1 / 32-29 mm	1 pc	MED1001070	4250201201898
Tandem Cube Pessary	3-2 / 37-32 mm	1 pc	MED1001069	4250201201904
Tandem Cube Pessary	4-3 / 41-37 mm	1 pc	MED1001068	4250201201911
Tandem Cube Pessary	5-4 / 45-41 mm	1 pc	MED1001067	4250201201928
Tandem Cube Pessary	6-5 / 55-45 mm	1 pc	MED1001094	4250201201935



Tandem Cube Pessary

Description

medesign's Tandem Cube Pessary consists of two connected cube pessaries and is used for severe vaginal and uterine prolapse. The concave surfaces of the Cube Pessary create an excellent vacuum. medesign's Tandem Cube Pessary can also help with stress incontinence. In this case, the bottom cube supports the urethra while the second cube ensures good retention.



Material

Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
Tandem Cube Pessary	0-0 / 25-25 mm	1 pc	MED1001634	4250201207227
Tandem Cube Pessary	1-1 / 29-29 mm	1 pc	MED1001635	4250201207234
Tandem Cube Pessary	2-2 / 32-32 mm	1 pc	MED1001636	4250201207241
Tandem Cube Pessary	3-3 /37-37 mm	1 pc	MED1001637	4250201207258
Tandem Cube Pessary	4-4 / 41-41 mm	1 pc	MED1001638	4250201207265
Tandem Cube Pessary	5-5 / 45-45 mm	1 pc	MED1001639	4250201207272



Tandem Cube Pessary

Description

medesign's Tandem Cube Pessary consists of two connected cube pessaries and is used for severe vaginal and uterine prolapse. The concave surfaces of the Cube Pessary create an excellent vacuum. medesign's Tandem Cube Pessary can also help with stress incontinence. In this case, the bottom cube supports the urethra while the second cube ensures good retention.



Material

Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
Tandem Cube Pessary	0-1 / 25-29 mm	1 pc	MED1001654	4250201207401
Tandem Cube Pessary	1-2 / 29-32 mm	1 pc	MED1001655	4250201207418
Tandem Cube Pessary	2-3 / 32-37 mm	1 pc	MED1001656	4250201207425
Tandem Cube Pessary	3-4 / 37-41 mm	1 pc	MED1001657	4250201207432
Tandem Cube Pessary	4-5 / 41-45 mm	1 pc	MED1001658	4250201207449
Tandem Cube Pessary	5-6 / 45-55 mm	1 pc	MED1001659	4250201207456



T Pessary Type B

Description

Pelvic floor defects in women are often a combination of cystocele, enterocele and rectocele. A cystocele occurs when the very thin wall between the bladder and vagina has become too weak, and the bladder partially sinks into the vagina. Thus a complete emptying of the bladder is no longer possible.

Especially during physical activity, such as heavy work or sport. The T Pessary supports the bladder and bowel in the anatomically correct position.



terial Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
T-Pessar Typ B	Size 1 / 20 mm	1 pc	MED1001649	4250201207357
T-Pessar Typ B	Size 2 / 25 mm	1 pc	MED1001640	4250201207289
T-Pessar Typ B	Size 3 / 32 mm	1 pc	MED1001650	4250201207364
T-Pessar Typ B	Size 4 / 36 mm	1 pc	MED1001727	4250201208002
T-Pessar Typ B	Size 5 / 42 mm	1 pc	MED1001728	4250201208019
T-Pessar Typ B	Size 6 / 46 mm	1 pc	MED1001729	4250201208026



T Pessary Type R

Beschreibung

Pelvic floor defects in women are often a combination of cystocele, enterocele and rectocele. A cystocele occurs when the very thin wall between the bladder and vagina has become too weak, and the bladder partially sinks into the vagina. Thus a complete emptying of the bladder is no longer possible.

Especially during physical activity, such as heavy work or sport, The T Pessary supports the bladder and bowel in the anatomically correct position.



Material

Medical silicone, Silcopure soft Transparent

Product name	Size	Contents	Article number	EAN-Code
T Pessary Type R	Ø 20 mm L 50 mm	1 pc	MED1001651	4250201207371
T Pessary Type R	Ø 25 mm L 55 mm	1 pc	MED1001641	4250201207296
T Pessary Type R	Ø 32 mm L 50 mm	1 pc	MED1001810	4250201208903
T Pessary Type R	Ø 32 mm L 62 mm	1 pc	MED 1001811	4250201208910
T Pessary Type R	Ø 32 mm L 72 mm	1 pc	MED1001812	4250201208927
T Pessary Type R	Ø 32 mm L 79 mm	1 pc	MED1001652	4250201208866
T Pessary Type R	Ø 36 mm L 50 mm	1 pc	MED1001813	4250201208934
T Pessary Type R	Ø 36 mm L 62 mm	1 pc	MED1001814	4250201208941
T Pessary Type R	Ø 36 mm L 73 mm	1 pc	MED1001815	4250201208958
T Pessary Type R	Ø 36 mm L 80 mm	1 pc	MED1001724	4250201208873



Product name	Size	Contents	Article number	EAN-Code
T Pessary Type R	Ø 42 mm L 55 mm	1 pc	MED1001816	4250201208965
T Pessary Type R	Ø 42 mm L 68 mm	1 рс	MED1001817	4250201208972
T Pessary Type R	Ø 42 mm L 76 mm	1 pc	MED1001808	4250201208880
T Pessary Type R	Ø 46 mm L 60 mm	1 pc	MED1001818	4250201208989
T Pessary Type R	Ø 46 mm L 73 mm	1 pc	MED1001819	4250201208996
T Pessary Type R	Ø 46 mm L 82 mm	1 рс	MED1001809	4250201208897

Incontinence Tampon Pessary Type C

The light silicone foam pessary



Female incontinence is a serious and disturbing problem for many women worldwide. The simplest therapy is to wear a supporting Incontinence Tampon Pessary which raises the front vaginal septum and reconstructs normal female anatomy.

The innovative medesign Incontinence Tampon Pessaries are manufactured from medical grade silicone foam. They can be used without any special preparation and have a high tensioning force which remains constant over the whole time of use – for the minimum of 3 months. The mainly closed pores of the foam and the hygienic properties of silicone allow easy cleaning and, if necessary, even disinfection.

Incontinence Tampon Pessaries are available in five sizes to fit the individual female anatomy. This optimal support restores continence in daily life.

As a medical device, Incontinence Tampon Pessaries are manufactured according to the medical device requirements and can be ordered directly from us.

es			

Involuntary loss of urine is a plight of many women. The cause is often a lack of pelvic floor muscles and / or a cystocele. The urine loss caused by sudden pressure on the bladder, such as sports, lifting, sneezing or laughing.

The Incontinence Tampon Pessary Type C provides a simple solution of the incontinence problem. By raising the anterior vaginal wall, the bladder neck is supported and mechanically repositioned so that the normal function of the bladder closure is restored in a natural way, thus preventing the unwanted urine leakage.

If properly adjusted, bladder emptying is not affected by the pessary. The vaginal secretion is retained and the normal vaginal flora is not impaired.



Material

Silicone foam

Product name	Size	Contents	Article number	EAN-Code
Incontinence Tampon Pessary Type C	Ø 20 mm L 60 mm	1 pc	MED1001900	4250201211163
Incontinence Tampon Pessary Type C	Ø 25 mm L 60 mm	1 pc	MED1001901	4250201211170
Incontinence Tampon Pessary Type C	Ø 34 mm L 70 mm	1 pc	MED1001898	4250201211194
Incontinence Tampon Pessary Type C	Ø 38 mm L 70 mm	1 pc	MED1001974	4250201212061
Incontinence Tampon Pessary Type C	Ø 45 mm L 80 mm	1 pc	MED1001902	4250201211187

Starting set for end users				
Incontinence Tampon Pessary Type C, Starting Set	Ø 25, 34, 38 mm	1 set	MED1001905	4250201211224

Dimensions



For design reasons, the length of the pessaries may differ by ± 3 mm.

Pessary Fitting Sets



Product name	Included sizes	Article number	EAN-Code
Cube Pessary, Fitting Set	29, 32, 73, 41 mm	MED1000487	4250201201874
T Pessary Type B, Fitting Set	Ø 25, 32, 36, 42 mm	MED1001733	4250201208071
T Pessary Type R, Fitting Set	Ø 25, 32, 36, 42 mm	MED1001732	4250201208064
Ring Pessary (thin), Fitting Set	65, 70, 75, 80 mm	MED1000489	4250201202581
Ring Pessary (thick), Fitting Set	65, 70, 75, 80 mm	MED1001834	4250201209313
Sieve Bowl Pessary, Fitting Set	65, 70, 75, 80 mm	MED1000656	4250201201492
Sieve Bowl Pessary EF, Fitting Set	65, 70, 75, 80 mm	MED1001835	4250201209320
Urethra Pessary, Fitting Set	65, 70, 75, 80 mm	MED1000765	4250201201621
Urethra Bowl Pessary, Fitting Set	65, 70, 75, 80 mm	MED1000488	4250201201706
Cramer Pessary, Fitting Set	60, 65, 70, 75 mm	MED1001770	4250201208521

Medesign fitting sets allow a individual adjustment of the pessary size in the doctor's office and contain 4 different sizes. They are made from medical grade silicone and are biologically tested. The fitting sets are made of sterilizable silicone and can be used on multiple patients. A detailed reprocessing instruction is enclosed.

Pessaries

For an effective treatment of pelvic organ prolapse (descensus) and incontinence

Pessaries are used for treating pelvic organ prolapse (descensus) and urinary incontinence. Gone is their reputation of causing infections, ulceration and pungent vaginal discharge. The old method of treatment has returned with the use of modern, more pleasant types of pessaries, which are made from a new kind of silicone, especially developed for pessary therapy.

This therapy is returning not least because of the long-term results of surgical treatments for descensus and incontinence, which are far from the operating surgeons' expectations.

For treating descensus and urinary incontinence, various pessary types have proven successful: The Urethra Pessary, the Cube Pessary, the Ring Pessary and the Sieve Bowl Pessary.

The Ring Pessary is used for treating descensus and stress incontinence or softening of scars (e.g. after plastic surgery of the vaginal wall), as well as for preparation prior to descensus surgery.

The Cube Pessary is used for treating uterine prolapse, descensus without urinary incontinence, a cystocele or

scars.

rectocele as well as a softening of scars.

The Urethra Pessary is placed on the medial parts of the levator ani. With this pessary descensus is

repositioned while the proximal urethra is moved ventrally and upwards, significantly improving pressure. However, if the levator ani is loose and insufficient, the Ring Pessary will not stay in place. In this case a women with stress incontinence should use the Cube Pessary.

This type of pessary works using a different principle, as cube pessaries adhere to the vaginal wall like suction cups, thereby being stabilised. Cube pessaries replace all other 'special constructions' of bygone days and effectively treat genital descensus and prolapse (with insufficient levator ani).

More than 20 years of experience of development and production of pessaries are our foundation for high and economic quality.

Uterine prolapse (Descensus uteri) is often caused by obstetric lesions or genetic weakness of the connective tissue. Uterine ligaments and pelvic floor muscles lose their ability to support the uterus. As a result, the uterus descends, causing a feeling of pressure and chaffing as well as back pain, ulcers and bleeding. A urethro-, cysto- or rectocele may often occur along with descensus. A urethrocele promotes stress incontinence as, when pushing, the urethra opens up funnel-like. A cystocele can bend the urethra (especially when prolapsed) and can result in micturition problems and urinary retention. A rectocele, consequently, causes defectation problems.

Should the Cube Pessary be used for repositioning a cystocele, occasional stress incontinence may occur. In this case one should switch to a Ring or Sieve Bowl Pessary. All patients with a cystocele, rectocele or descensus uteri and prolapse should be treated with cube pessaries prior to considering surgery.

The smallest adhering cube is chosen and can usually be replaced by an even smaller cube after the first or second month of treatment. After three to five months a permanent repositioning (without an inserted pessary) is achieved in more than 1/3 of cases – making surgery unnecessary. However, if satisfactory treatment of the condition and permanent repositioning is not achieved, despite bladder- and pelvic floor exercise as well as local oestrogen therapy, surgery cannot be avoided.

All patients with a cystocele, rectocele and uterusdescensus or -prolapse should only be treated with cube pessaries before surgery is considered.

Life-long pessary therapy is certainly possible. Patients remove the Cube Pessary at night and insert it again in the morning, using Ladysoft lubricant. This assures hygiene as well as relief from strain on the tissue.



»Passion for detail often makes the difference in pessary therapy.«

With kind regards

lothar engel

Application Pessaries	Area	Prolapse 1-2 degree 3-4 degree		Cystocele	Rectocele	Stress Incontinence
Ring Pessary (dünn)	0	\checkmark		\checkmark	\checkmark	\checkmark
Ring Pessary (dick)	0		\checkmark	\checkmark	\checkmark	\checkmark
Sieve Bowl Pessary		\checkmark	\checkmark	\checkmark	\checkmark	\checkmark
Sieve Bowl Pessary EF	0	\checkmark	\checkmark	\checkmark	\checkmark	\checkmark
Urethra Pessary	\bigcirc	\checkmark		\checkmark		
Urethra Bowl Pessary	0	\checkmark		\checkmark		

For more information visit our website: www.pessartherapie.de

Cerclage Pessary	0	To avoid premature delivery in cervical insufficiency.
Cerclage Pessary H & J	δ	to avoid premature delivery in cervical insufficiency.

Application Area Pessaries		Prol 1-2 degree	apse 3-4 degree	Cystocele	Rectocele	Stress Incontinence
T Pessary Type R		\checkmark		\checkmark	\checkmark	\checkmark
T Pessary Type B		\checkmark		\checkmark	\checkmark	\checkmark
Cube Pessary		\checkmark	\checkmark	\checkmark	\checkmark	\checkmark
Tandem Cube Pessary		\checkmark	\checkmark	\checkmark	\checkmark	\checkmark
Cramer Pessary	\mathbf{O}	\checkmark		\checkmark		\checkmark
Cerclage Pessary H & J	Ø		\checkmark	\checkmark	\checkmark	
Club Pessary	8		\checkmark			

Incontinence Tampon Pessary Type C			\checkmark	\checkmark	\checkmark
---------------------------------------	--	--	--------------	--------------	--------------

		Sizes	Silicone	Silicone foam	Silicone and Metal
Ring Pessary (dünn)	0	45 – 100 mm			\checkmark
Ring Pessary (dick)	0	50 – 100 mm	\checkmark		
Sieve Bowl Pessary	0	50 – 100 mm	\checkmark		
Sieve Bowl Pessary EF	0	50 – 100 mm	\checkmark		
Urethra Pessary	\bigcirc	45 – 100 mm			\checkmark
Urethra Bowl Pessary	0	55 – 85 mm	\checkmark		

For more information visit our website: www.pessartherapie.de

Cerclage Pessary	0	32/65/H17 mm 35/70/H25 mm	\checkmark	
Cerclage Pessary H & J	Ø	30/L57/B59/H14 mm 35/L80/B83/H14 mm	\checkmark	

		Sizes	Silicone	Silicone foam	Silicone and Metal
T Pessary Type R		20/L50 – 36/L80 mm	\checkmark		
T Pessary Type B		20/L53 – 46/L81 mm	\checkmark		
Cube Pessary		25 – 45 mm	\checkmark		
Tandem Cube Pessary		25 – 55 mm	\checkmark		
Cramer Pessary	O	50 – 90 mm			\checkmark
Club Pessary	8	50 – 90 mm	\checkmark		

Incontinence Tampon Pessary Type C		20/L60 – 45/L80 mm		\checkmark	
---------------------------------------	--	--------------------	--	--------------	--

Complementary Products

Ladysoft Lubricant

Balances natural moistness of the vagina in a pleasant way and makes intercourse enjoyable. Experienced gynecologists recommend Ladysoft to their patients as lubricant, it's fat-free and can be that can be used safely over a long period. It does not interfere with the endocrine and is kind to the skin, as it is not like other fat-containing lotions and will not adhere to the mucosa or cause skin irritation.







All medesign pessaries are delivered in a user-friendly set.

- 1 medesign pessary
- 1 Ladysoft lubricant
- 1 Instructions for use

Made in Germany

All pessaries are manufactured in our production facility – providing a solid basis for consistent high quality and innovation.

Complementary Products

Femsoft Intimate Washing Lotion

Especially developed for daily care of the genital area. The low pH-value of 3.8 - 4.0 naturally prevents growth of bacteria.

Natural substances and herbal extracts strengthen the flora of the genital area. Should be gently rubbed in for about 2 to 3 minutes for the wash to be effective. Suitable for daily care.

Femsoft Intimate Care Lotion

Femsoft Intimate Lotion nourishes the skin and protects its natural acid mantle. The natural balance of the delicate skin in the intimate area is protected and feels soft and velvety. Even itching of the anal- or vaginal area can be soothed. This product is also recommended when physically very active. Recommended by gynaecologists and proctologists. Massage a small amount of Femsoft Intimate Lotion into the dry and clean skin of the entire intimate area and a protective moisturizing film will be formed.

Femsoft Intimate Care Cream

Femsoft Intimate Cream protects the intimate area when especially stressed. It alleviates discomfort and regenerates and balances vaginal flora. Furthermore, it protects against infections. The properties of Femsoft Intimate Cream protects the skin by forming a protective film and thereby smoothing and calming the wounded, stressed skin. By concealing skin injuries, bacteria and fungus cannot grow. The sequence of contamination, washing and irritation is interrupted.

Femcon Plus Vaginal Cones

Active training of the pelvic floor muscles with Femcon Plus Vaginal cones can effectively cure apparent stress incontinence.

Femcon Plus consists of a set of five tampon-like vaginal cones which ascend in weight. After insertion into the vagina they have to be held in place by contracting the pelvic floor. A nylon string ensures easy removal after training.









Service

Short delivery times

We have all pessary types and sizes in stock. Orders, that reach us by 12.00 a.m. (midday) are shipped on the same day as parcel post.

Expert service

We will advise you by phone, email or through our contact form on the website.

Further information - www.pessartherapie.de

You may sign up on our pessary info page to be included in our data bank for physicians.

Own development and construction

All moulds for our pessaries are developed and produced by medesign. The process of manufacture is adapted specifically for the specially developed medical silicone.

Webstore - www.medesign.de

You have the possibility to shop in our webstore comfortabley at home. Aside from pessaries, you can find a whole range of products from different categories.

medesign I.C. GmbH

- Dietramszeller Str. 6 D-83623 Dietramszell Germany
- +49 (0)8027 90738-0
- +49 (0)8027 90738-27
- @ info@medesign.de



medesign I.C. GmbH

Dietramszeller Str. 6 · D-83623 Dietramszell · Germany · info@medesign.de · www.medesign.de